



# THE CRYOTEC METHOD





# Our goal is to provide all patients the reliable and effective vitrification technique.

All of the necessary materials and methods to achieve this can be found in these two packages alone:

"Cryotec<sup>®</sup> Vitrification Kit 101" "Cryotec<sup>®</sup> Warming Kit 102"

We hope that all infertility patients around the globe can share in the joy of these packages filled with the miracle of science.





# **Table of Contents**

1. Basic Operation	•••P. 4
Microscope Magnification	•••P. 4
Position of the Oocyte/Embryo During Aspiration	•••P. 4
2. Vitrification Protocol	•••P. 5
Materials	•••P. 5
Vitrification Preparation	•••P. 5
[STEP 1] Equilibration via ES	•••P. 6
[STEP 2] Equilibration via VS1	•••P. 9
[STEP 3] Shrinkage via VS2	•••P. 11
[STEP 3] Placing the Oocyte/Embryo	•••P. 12
[STEP 3] Ultra-Rapid Freezing	•••P. 12
3. Warming Protocol	•••P. 14
Materials	•••P. 14
Warming Preparation	•••P. 14
[STEP 4] Warming via TS	•••P. 15
[STEP 5] Dilution via DS	•••P. 17
[STEP 5] Dilution via WS1	•••P. 18
[STEP 6] Washing via WS2	•••P. 19



# 1. Basic Operation

#### [Microscope Magnification]

. . . . . . . . . . . . .

The Cryotec Method uses only two magnification levels of the microscope for a simple operation.



### [The Position of the Oocyte/Embryo During Aspiration]

Using Cryotec Method protocols, there are only three positions for the oocyte/embryo that you will have to be familiar with.

① Aspirating a small amount of the solution (roughly double the diameter of the oocyte) after the oocyte/embryo (standard procedure).



2 Placing the oocyte/embryo itself at the tip of the pipette (equilibration via vitrification).



③ Aspirating 3mm of solution after the oocyte/embryo (dilution via TS and DS).



#### Tip!

All actions involving the pipette and oocyte/embryo can be made significantly easier by using capillary action (a natural occurrence) to aspirate 1mm of solution into the pipette beforehand. To make things even easier, you can make a mark on the pipette to indicate the threshold at which the capillary action ends.





# 2. Vitrification Protocol

#### The purpose of cryopreservation via vitrification

Using cryopreservation via vitrification prevents any change or deterioration in the quality of the oocyte/embryo, ensuring that it preserves the same state it was in before freezing.

# [Materials]

- Cryotec® Vitrification Kit 101
  - Equilibration Solution (ES): 1.0 ml/vial×1
  - Vitrification Solution (VS): 1.0ml/vial×2
  - 4 Cryotecs
  - 3 Vitri-plates with 3 wells each (and lid)
- Microscope (with heating stage disabled)
- Timer (with count-up function)
- Tweezers
- Scissors
- Micro-pipette (-300µl)
- Pasteur pipette (with mouth piece)
- **Stripper (with stripper tips)**
- Cooling rack



#### Vitri Plate Characteristics

The vitrification plate, specially designed for use with the Cryotec Method, has 1) rounded wells, 2) space for depositing solution expelled during washing, and 3) a groove for securely holding the Cryotec (carrier device).

Because each well is hemispherical, they are less likely to create "blind spots" due to shadows in the microscope. This makes it much more difficult to lose track of the oocyte/embryo.

#### [Vitrification Preparation]

- Maintain the room temperature between 25°C and 27°C. 1.
- 2. Bring out the ES and VS vials to room temperature  $(26\pm1^{\circ}C: 25-27^{\circ}C)$  at least 1 hour before performing this protocol.

(CHECK: Examine the vials by inspecting visually and touching with your fingers before using them to confirm the presence of any abnormalities, such as unusual colorations or temperatures higher/lower than expected.)

- Prepare the manipulation pipette with an inner diameter that matches the diameter of the oocyte/embryo to be 3. vitrified:
  - For oocytes/embryos, <u>140-150µm</u>.
  - For blastocysts, 160-220µm.

4.

- In the event that the diameter of the blastocyst to be vitrified is greater than 220µm, conduct the following preshrinking process:
  - Immerse the blastocyst in hypertonic solution (DS:WS = 2:3) until it reaches the proper size. This is the most effective and non-invasive contraction method.





### [**STEP** 1]

#### **Equilibration in ES (8-15 minutes: at room temperature)**

The purpose of this step is to introduce cryoprotectant agents (CPAs) into the cell, the successful completion of which is indicated by the complete recovery of the cell's volume.

#### The mechanism behind cell shrinkage and recovery

Because the inside of the cell is a culture medium (with an osmotic pressure of approximately 300) and the extracellular space is filled with ES (with an osmotic pressure of approximately 2,400), water within the cell will flow out of it due to this difference between intra- and extracellular osmotic pressures. At the same time, because CPAs can penetrate the cell membrane, they flow into the cell. Because the intra- and extracellular osmotic pressures naturally attempt to equilibrate, these reactions occur simultaneously. However, the rate at which water flows out of the cell is slightly faster than that of CPAs flowing into the cell. This means that the oocyte/embryo will first shrink as the water leaves, then return to its original volume as the CPAs enter.





#### CHECK: Avoid forming bubbles when dispensing.

Attach the chip to the 300  $\mu$ l micropipette. Aspirate 300  $\mu$ l of solution into the pipette and expel it inside of the vial. Then aspirate again the solution into the pipette and slowly dispense it. This will avoid the unnecessary formation of bubbles.

#### **CHECK:** If bubbles form, let the solution sit.

If bubbles do form in the solution, these will disappear if you simply leave it for a while. Do not attempt to remove the bubbles with the micropipette, as this will change the amount of solution within the well.





**CHECK:** Note that once the oocyte/embryo is placed in the ES, its volume will begin shrinking immediately. Please confirm the shrinkage; if it doesn't shrink, the oocyte/embryo is not viable.



#### Evaluating oocyte/embryo quality based on shape during shrinkage

You can gauge the quality of the oocyte/embryo by the membrane permeability of its cell. A high-quality oocyte/embryo will maintain its spherical shape while shrinking (Rank A). If there is variation in the membrane's permeability, irregular edges will form (Rank B), and as the quality decreases it will display an extremely non-uniform shape (Rank C). <u>Please note that these grades do NOT affect survival rate;</u> however, because this represents the quality of the oocyte/embryo itself, it may indicate a difference in fertility, developmental ability, or pregnancy rate at a later stage.







- While waiting for the oocyte/embryo to recover its volume, remove the straw containing the Cryotec from the package and prepare the Cryotec. Fill in the oocyte/embryo information on the back side (opposite of the logo) and place the Cryotec in the groove on the Vitri Plate, making sure to keep the logo facing up (1).
- 2. Prepare the fresh liquid nitrogen in the cooling rack.
- **3.** Once the oocyte/embryo has fully returned to its original size, the **ES** equilibration is complete.

#### **Determining equilibration completion**

If volume recovery cannot be confirmed, or if you are unsure in your judgment of complete recovery, the maximum completion times for ES equilibration are as follows: Oocytes and blastocysts ( $160\mu m - 220 \mu m$  in diameter) – 15 minutes; 4-8 cell stage embryos – 12 minutes. It has been conclusively determined that sufficient equilibration occurs after 15 or 12 minutes, respectively.



It can sometimes be difficult to determine whether a blastocyst's volume has completely recovered due to the blastocyst cavity. Therefore, in the case of blastocysts, inspect the contact between the zona pellucida and TE as shown in the figure above and determine recovery accordingly. Viewing the blastocyst from the top, imagine a cross overlaid across the cell; if the zona pellucida and TE are in contact with each other at 3 out of the 4 points which intersect with the perivitelline space, the blastocyte has sufficiently recovered.





# [STEP 2]

#### Equilibration in VS1 (30-60 seconds)

The purpose of VS1 is to replace all extracellular ES with VS. This step is complete when the densities of the cell and VS1 are equal.



#### **CHECK:**

Because the released oocyte/embryo will quickly float to the surface, your microscope's focus should be set on the liquid's surface, allowing you to confirm the presence of the oocyte/embryo.



#### **CHECK:**

Use high magnification on the microscope to confirm that the oocyte/embryo has completely stopped in the middle of the solution. Make sure that the oocyte/embryo remains in focus for a while, confirming that it has stopped floating.







#### **CHECK:**

The purpose of aspirating fresh VS2 before aspirating the oocyte/embryo from VS1 is to prevent bringing VS1 into VS2.







### [STEP 3]

#### Shrinkage in VS2 (10-20 seconds)

The purpose of VS2 is to confirm that ES has been completely replaced by VS. This is confirmed in two ways: 1) The oocyte/embryo shrink completely (into a moon/kidney/bean shape), and 2) the oocyte/embryo is not floating. (It means that there is no more ES inside)



#### **CHECK:**

If you can observe the oocyte/embryo's shrinkage by naturally, there is no need to stir it.





#### Placing the Oocyte/Embryo



**CHECK: 1 oocyte = 1 drop. DO NOT REDUCE the volume of the droplet.** 

#### Do not attempt to reduce the size of the droplet

Conventional methods would attempt to minimize the size of the droplet in order to increase the cooling rate. However, since the freezing reagent used in the Cryotec method has higher vitrification ability than these conventional methods, there is no need to do this. In attempting to minimize the droplet, you may apply pressure to the oocyte/embryo due to surface tension. This may damage the oocyte/embryo, or cause it to stick to the sheet, making it difficult to remove during thawing. This can also increase the time required for TS, which may damage the oocyte/embryo. To reiterate, our method does NOT require you to minimize the droplet. If the droplet is overly large, or if two or more oocytes/embryos are placed into a single droplet, simply re-aspirate it back into the pipette and make a new droplet at a different location on the sheet.

#### **Ultra-Rapid Freezing**





#### Attaching the Cap





# 3. Warming Protocol

#### **Temperatures at the risk of forming ice crystals**

The thawing process is the most likely stage at which ice crystals may form. Even with an increased cooling rate during vitrification, ice crystals may still form if the heating rate is too low. It is extremely important to move quickly from -80 °C to -20 °C, which is the range in which ice crystals are most likely to form.

#### [Materials]

- Cryotec® Warming Kit 102
  - Thawing Solution (TS) 1.8 ml/vial×1
  - Diluent Solution (DS) 0.5 ml/vial×1
  - Washing Solution (WS) 1.0 ml/vial×1
  - 1 Warm Plate with 4 wells
- Microscope (with heating stage disabled)
- Timer (with count-up function)
- Tweezers
- Micro-pipette (-300µl)
- Pasteur pipette (with mouth piece)
- **Stripper (with stripper tips)**
- Cooling rack



#### Special thawing plate characteristics

Our Warm Plate, designed exclusively for its use with the Cryotec Method, features (1) a rectangular well inclined for TS use. It also includes (2) hemispherical wells and (3) space for expelling solution, similar to the our Vitri Plate. The slope of the TS well is designed to allow stable placement of the Cryotec sheet. The hemispherical wells allow easier making of a liquid layer of the solution during dilution and makes it possible to replace gradually the osmotic pressure.

### [Warming Preparations]

Use an incubator to heat the Warm Plate and TS vial (with cap closed) to 37 °C at least 2 hours prior to beginning 1. the thawing process (warming overnight is acceptable).

• When using a CO<sub>2</sub> incubator, make sure that the TS vial cap is tightly secured before introducing it into the incubator (as above, overnight warming is acceptable).

- 2. Keep DS and WS at room temperature (26±1°C: 25-27°C) for at least 2 hours prior to thawing.
- 3 Prepare the liquid nitrogen in the cooling rack. Remove the Cryotec from the liquid nitrogen tank and place it in the cooling rack. Inside the liquid nitrogen remove the cap and place the Cryotec against the inside wall of the cooling rack.



#### [STEP 4] Warming in TS (1 minute)



#### **CHECK:**

In order to ensure the maximum temperature of 37°C, it would be best if all room-temperature solutions were placed immediately before they're required (since they would reduce the overall temperature of the Warm Plate). However, because this step only takes 1 minute, there isn't enough time to do this, which is why DS is dispensed in advance.





Immerse the Cryotec in the TS and let it sit undisturbed for a full minute. If you move it, it can upset the temperature balance of the entire solution due to the extremely low temperature of the Cryotec. In addition, if the oocyte/embryo has come loose from the sheet, any movement may displace it and cause you to lose sight of it. If you are unable to see the oocyte/embryo on the Cryotec sheet, remain patient and do not attempt to move the sheet.



#### Tips for inserting the Cryotec into the TS well

Prepare the pipette using your right hand and hold the Cryotec in your left hand, keeping your grip as close to the sheet as possible. Immediately insert it at an angle of 30 to 40 °, using the curve of the TS well to guide you. This may cause your hand to unconsciously arc, so it's important to make an effort to move your hand in a short, straight line. You can also place the side of your hand on the stage first, which will keep it more stable as you insert the Cryotec. Be sure to move your hand quickly from the liquid nitrogen to the stage of the microscope, then carefully and gently insert the Cryotec sheet into the TS well. Move as fast as possible between the liquid nitrogen and TS using the curve of the TS well, to avoid making any bubbles on the sheet. If any air bubbles form on the sheet, it may make it more difficult to find the oocyte/embryo, or the oocyte/embryo may stick to a bubble and move within the solution. Please exercise all necessary caution.





#### [STEP 5] Dilution in DS (3 minutes)

	Figure	Procedure
1	1 2 3mm TS	<ol> <li>Let the Cryotec sit for 1 full minute, then slowly remove it from the TS well (1).</li> <li>Aspirate the oocyte/embryo within the TS, then slowly aspirate 3mm of TS into the pipette over 3 seconds (2).</li> </ol>
		-

#### **CHECK:**

If the oocyte/embryo has still not separated from the sheet after a full minute, place the Pasteur pipette beneath it and apply gentle pressure to separate it. Be sure not to come into direct contact with the oocyte/embryo.

#### **CHECK:** If the oocyte/embryo disappears

If you successfully place the oocyte/embryo on the Cryotec sheet during the vitrification process, the oocyte/embryo is in the TS well. Our TS solution has minimal toxicity, so you will have time to patiently search for the oocyte/embryo.

#### **CHECK:**

You can measure out 3mm using the indicator on the Warm Plate lid.





#### **Dilution in WS1 (5 minutes)**



#### **CHECK:**

It's important to confirm during this step whether or not the oocyte/embryo was damaged during the vitrification/thawing process. An oocyte/embryo which has survived will undergo a normal membrane reaction and subsequent volume recovery.

The reaction during this stage is due to condensation: The oocyte/embryo becomes isotonic as it enters WS (300) from DS (900). In other words, while the volume of a fully intact oocyte/embryo will have recovered completely, a lower quality oocyte/embryo will need more time to do so. In the case of a dead or damaged oocyte/embryo, normal membranous reaction will not occur at all. As such, you will see no change in volume whatsoever. In the case of blastocysts, once the blastocyst cavity begins to form/expand, or the blastocyst cavity is completely re-expanded, it can be considered to have survived. In general, it has been shown that human embryos are likely to become children if more than 30% of blastomeres survive:

- For a 2-cell stage embryo, 1 blastomere
- For a 4-cell stage embryo, 2 blastomeres
- For an 8-cell stage embryo, 3 blastomeres



#### Washing in WS2 (1 minute)



#### **CHECK:**

We recommend culturing the oocyte for 2 hours for ICSI and the blastocyst for at least 1 hour before embryo transfer.