



CRYOTEC IS EVOLVING WITH Ready to Use

NEW OOCYTE AND EMBRYO VITRIFICATION AND WARMING METHOD


5 REASONS TO CHOOSE THE CRYOTEC RtU METHOD

1. Convenient

By pre-dispensing solutions in plates, the new high-quality blister pack design effectively minimizes handling and saves preparation time.

2. Time saving and effective

Vitrification and warming protocols maximum time is reduced by 5.5 minutes due to the qualities of the new solutions.

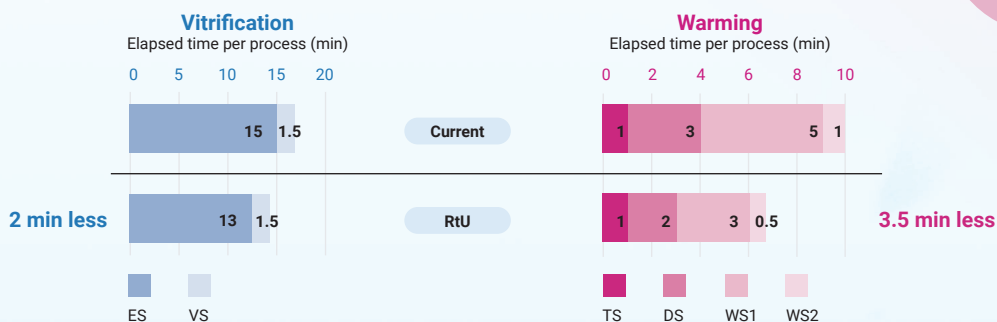

CREATED WITH
EMBRYOLOGISTS
IN MIND

NO
PREPARATION
REQUIRED

TOTAL OF
5.5 MIN
SAVED*

3. Only one protocol for oocytes and embryos of all stages

With Cryotec solutions, a single protocol ensures the safety of oocytes and embryos, promoting standardized lab practices.

4. Cost and waste effective

RtU reduces medical waste by minimizing the dispensing process, resulting in cost and inventory reduction.

5. Embryologist-friendly

The new Cryotec RtU products are designed with the embryologist in mind, offering a stress-free experience with no strict time limits.

*According to our research




RtU FEATURES

IMPROVED VITRIFICATION AND WARMING PLATES' DESIGN



Labeled wells with the solution name to avoid mistakes

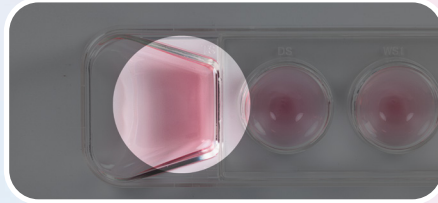

Redesigned shape of the TS well aids precise insertion of the sheet



3mm indicator on the warming plate as a reference for accurate aspiration of the solutions



Laser AHA compatibility enables embryologists to perform AHA using the same RtU plate with the new warming plate design



Sloping bottom of TS well helps to avoid bubble formation when inserting the Cryotec into TS well

NEW SOLUTIONS FORMULA

Promoted CPA permeability improves vitrification performance, reducing processing time by 5.5 minutes.*

No precipitation under incubator temperature eliminates the need for separate incubation of warming solutions and plates.

Higher viscosity facilitates easier oocyte/embryo manipulation and lowers the risk of osmotic shock.

EACH PACK CONTAINS

Ready to Vitri kit



- 3 vitrification blister packs
- 4 Cryotecs

Ready to Warm kit



- 5 warming blister packs

HOW TO ORDER

Please contact us by email globalsales@reprolife.jp or scan QR code to visit our contact page



*According to our research

Note: This brochure contains information about various studies related to Cryotec products. This information is only a summary provided for information purposes about Cryotec products. This information is provided "as is" without any warranties, expressed or implied, including but not limited to the implied warranties or suitability or eligibility for any particular purpose or success of treatment on an individual basis. Products and information may have changed since the printing of this brochure. For more information see our website www.cryotec.com

This flyer was updated in July 2023

Produced by REPROLIFE Inc.

